



**01** Take the tube of **Silken**<sup>®</sup>, remove the cap and puncture the foil seal using the point on the reverse side of the cap.

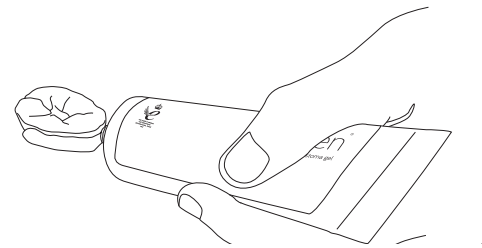


**02** Squeeze the silicone gel from the tube directly onto the skin to fill any depressions and skin contours.




**03** Apply your pouching system in the normal way directly onto the wet gel. You don't have to wait for the gel to set.

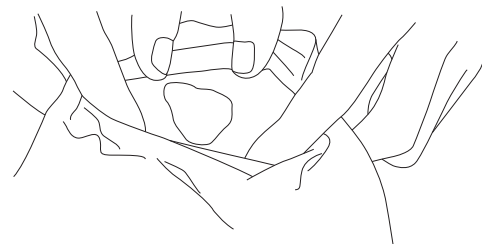

**04** The gel will cure/set under the pouch during normal wear time. Unlike other pastes, **Silken**<sup>®</sup> remains flexible and will be very comfortable in use.

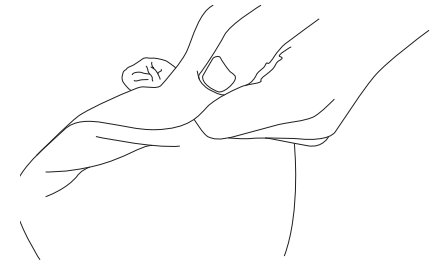

**05** To remove the gel, gently peel the wafer of your pouch from the skin as normal. Usually the gel will be stuck to the wafer and will peel away with it. If not, it will be very easy to peel off the gel in one piece after use. It won't leave any residue on your skin.

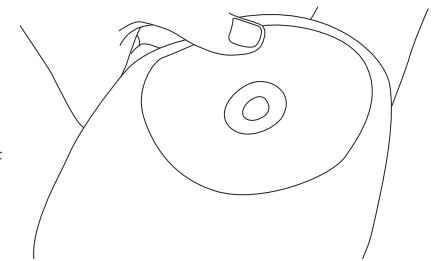

Always make sure your skin is clean and dry before applying **Silken**<sup>®</sup>, as it will not adhere to wet skin. **Silken**<sup>®</sup> can be used to fill uneven skin or areas that are prone to leakage.